



**US
DENTAL
DEPOT**

US DENTAL DEPOT

6555 NW 9th Avenue suite 212
Fort Lauderdale 33309
Florida USA

USA - 1 1866 625 5558 – 1 954 874 6325

Colombia 57 1 489 9051

Peru 51 1 707 3444

Venezuela 58 212 729 1639

Argentina 54 11 5258 7148 Cba 54 351 569 1409

www.USDENTALDEPOT.com

PLASTER TRIMMER

FOR PROSTHESIS/ORTHODONTICS W/ POLISHER # 100892

How to use:

It is used to trim and finishes cast models.

Plaster trimmer characteristics:

- Frontal cover in aluminum,
- Cast does not adhere on the equipment body; it is rust free making cleaning easier.
- 10" grind stone, easy to replace (can be used with both sides)
- Working area allows trimming two models.
- It comes with rubber base to best absorb vibrations and smoothly trimmer models.
- The water flux is distributed all over the grind wheel avoiding the cast to adhere and also avoiding particles in suspension.
- To substitute the grind wheel it is necessary to loose four nuts located on the front of cover (without using keys)
- All the nuts and screw are made of brass to prevent from oxidation.
- Includes an orthodontic angulating working table with a device for horizontal trimming.
- Its painting is made in epoxy to provide higher durability and protection against cast.

Technical characteristics:

Gross weight: 13,80 kg

Potency: ½ hp

Frequency: 50/60 Hz

Rotation: 1,750 rpm

Consumption: 368 watts

Tension: 110/220V

Size: width : 33 cm ; height; 50cm length ; 50 cm

Advantages:

It can be used for both prosthesis and orthodontics, and it has a polisher.



**US
DENTAL
DEPOT**

US DENTAL DEPOT

6555 NW 9th Avenue suite 212
Fort Lauderdale 33309
Florida USA

USA - 1 1866 625 5558 – 1 954 874 6325

Colombia 57 1 489 9051

Peru 51 1 707 3444

Venezuela 58 212 729 1639

Argentina 54 11 5258 7148 Cba 54 351 569 1409

www.USDENTALDEPOT.com

Discs with a double face and easy to substitute.
Electric system fully isolated
Easy installation

Instruction to installation and use

Link the socket in the electric net, verify if the voltage of the net is compatible with the equipment.

WARNING: The equipment is BIVOLT, but it comes from factory adapted to operate in 220V, to change the voltage you must to alter the key position.

Fix the table screwing the two screw at the frontal cover(you'll have to use a screwdriver) then it will be ready to be used in prosthesis.

To use the table to prosthesis and orthodontics you will have to follow these instructions:

- To change the table to orthodontics pull back the plastic nuts.
- Pull back the articulator of the table.
- Pull back the screw which regulates the horizon table.
- Introduce the angle to orthodontics clipping.

Installing the outlet water:

- Down view connect $\frac{3}{4}$ " hose to water outlet.
- Fit the table tighten the screw until the end
- Frontal view : Be careful to open the tap just enough to wet the disc.
- Connect to water inlet using $\frac{1}{2}$ " hose.

Warranty

The Essence Dental guarantees its equipment and respective accessories against material defects and man power, since that such equipment is operated in agreement with the conditions of specified use and operation in the respective Manual of Instructions. The total stated period of guarantee is described in "benefits" of the Manual of Instructions and valley of the date where the equipment was acquired.

Excluded items of the warranty



**US
DENTAL
DEPOT**

US DENTAL DEPOT

6555 NW 9th Avenue suite 212
Fort Lauderdale 33309
Florida USA

USA - 1 1866 625 5558 – 1 954 874 6325

Colombia 57 1 489 9051

Peru 51 1 707 3444

Venezuela 58 212 729 1639

Argentina 54 11 5258 7148 Cba 54 351 569 1409

www.USDENTALDEPOT.com

This warranty has not covered:

- Defects or resultant damages of use of the equipment in another way that not specified it in the respective Manual of Instructions;
- Defects or decurrent damages of repair, disassemble, tests, installation, alteration or any type of modification carried through for people or assistências not authorized techniques for the Dental Essence;
- Any accessory or component subject to the natural consuming during the use of the equipment (*cloth light bulb, refill, filters, cloth gloves, electrodes, ribbon amarlon and record of cut*);
- Equipment that has had the serial number removed or become unreadable;
- Scratches, fictions, or any another actual damage to the equipment, in reason of the diverse use of the specified one in the Manual of Instructions;
- Defects or actual damages for fall of the equipment, recklessness, accidents, electric discharge or discharge in the electric net.

WWW.USDENTALDEPOT.COM