



**US
DENTAL
DEPOT**

US DENTAL DEPOT

6555 NW 9th Avenue suite 212
Fort Lauderdale 33309
Florida USA

USA - 1 1866 625 5558 – 1 954 874 6325

Colombia 57 1 489 9051

Peru 51 1 707 3444

Venezuela 58 212 729 1639

Argentina 54 11 5258 7148 Cba 54 351 569 1409

www.USDENTALDEPOT.com

Instructions Manual *Trijato Gold Line*

Use:

Equipment to be used on the cast pieces cleaning and sandblasting to improve and facility its adherence (pressed ceramic and pieces requiring micro-retention to facilitate or improve the adherence)

The Alumina, Glass Sphere or Special Sand shall be used for:

- Micro-retentions on the pieces internal surface to be cemented;
- Cleaning the orthodontic brackets;
- Removal of the internal and external oxidation in the pieces to be cemented;
- Removal of the provisory and definitive cements;
- Superficial degreasing of the prosthetic piece;
- Fine adjustment of the occlusal area;
- Sandblasting in ceramic.

Benefits:

- Manufactured in plastic. It is not liable of oxidation, different from the cabinets manufactured in steel plates. It also does not become yellowish or dries as the high-impact polystyrene (which is not recommended for this product type).
- Revolutionary design having front access for the hands and a wide 45°-angle panel, making it easier to view the piece and providing the correct position to the operator, thus avoiding the muscular fatigue.
- Gloves support ring easily replaced.
- Protection film avoids the glass obfuscation.
- It has an optimum sealing on panel.
- It has two tanks up to 2 kg capacity each.
- It has directional valve to select the blasting nozzle.
- On/off switch with protection to avoid the damage by the blasting particles.
- It has air condenser filter thus avoiding the humidity in the abrasive element.
- The modern and ergonomic design developed for the hand pieces in order to optimize the professional work. The refills are manufactures in special steel (videa) ensuring a high durability.
- It has an exhauster for the sandblasting particles, which hinder the pieces view during the work thus eliminating the need to wear masks to protect the operator breathing system.
- It has fluorescent lighting system.
- VH line has 18 months warranty against manufacture defects and low cost for spare parts.



**US
DENTAL
DEPOT**

US DENTAL DEPOT

6555 NW 9th Avenue suite 212
Fort Lauderdale 33309
Florida USA

USA - 1 1866 625 5558 – 1 954 874 6325

Colombia 57 1 489 9051

Peru 51 1 707 3444

Venezuela 58 212 729 1639

Argentina 54 11 5258 7148 Cba 54 351 569 1409

www.USDENTALDEPOT.com

Installation and Use Instructions:

- Plug to the electrical system and verify if the network voltage is compatible to the one selected in the equipment.

REMARKS: This is two-voltage equipment, but is adjusted to operate in 220 v. Change the switch position to change from 220 v to 110 v.

- Connect the air input hose to the pneumatic network and adjust the air filter to work between 80 and 100 ounces.
- Put oxide, sand or glass sphere into the respective tanks.
- Set the on/off switch to use the exhauster and lamp.
- Turn the knob to select the nozzle to be used and press the pedal.

Technical Features:

Net Weight: 5 kg

Gross Weight: 7 kg

Use Pressure: 80 up to 100 lb

Electrical Fan: Two-voltage 0.26 A / 0.11 A

Electronic Reactor: 8 watts

Fluorescent Lamp: 8 watts

Tension: 110V/220V

Frequency: 50/60 Hz

Consumption: 45 watts

Dimensions: 44 cm (length), 44 cm (wide) and 31 cm (high)